



INIBLANC

Lift up your value

A new type of Injectable for lifting the entire Face

KEY TECHNOLOGY

CESABP Technology by DEXLEVO INC.

DEXLEVO's Collagenesis-Enabled Solubilized Active and Biodegradable Polymer Technology is a unique technology that enables regeneration of natural skin collagen without the use of microparticles.

World's unique technology of manufacturing of Solubilized PCL for natural skin collagen synthesis



Patent no.(KR) : 10-2018-0058947



Patent no.(US) : 16/410,611



Patent no.(PCT) : PCT/KR2018/005988

Key Features of INIBLANC

Premium Collagen Stimulator

Fully solubilized liquid form of Polycaprolactone (PCL)

DEXLEVO
505, Daeryung INC.
Seoubusaet-gil, Geumcheon-gu, Seoul, Korea
TEL 82-2-2138-8780 FAX 82-2-2138-8781

INIBLANC
Injectable Implants, PCL

WHY INIBLANC

01

THE 'REAL' LIQUID FACELIFT

Revolutionary anti-aging
effect for facial lift


02

ENCOURAGE SKIN ELASTICITY

Repair loss of skin elasticity
by Neo-Collagenesis



NEO-COLLAGENESIS

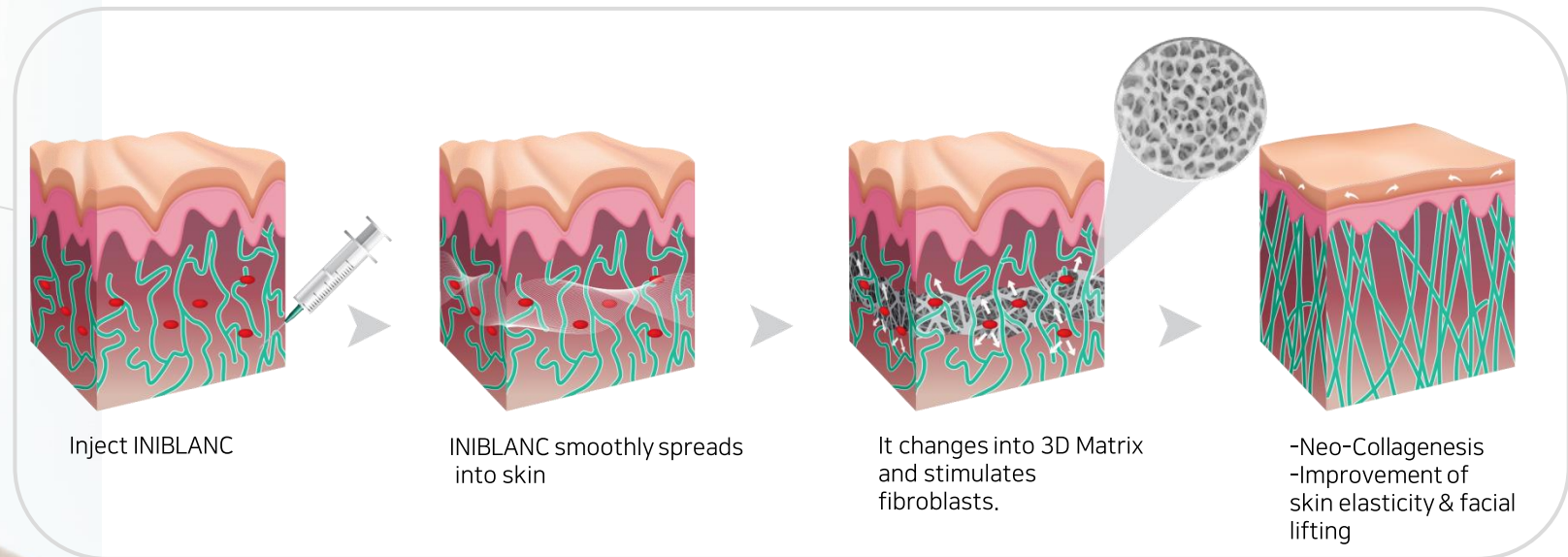


FULLY SOLUBILIZED PCL

/ Competitor /



Polymer microparticle + carrier

Injection 26~27G needle

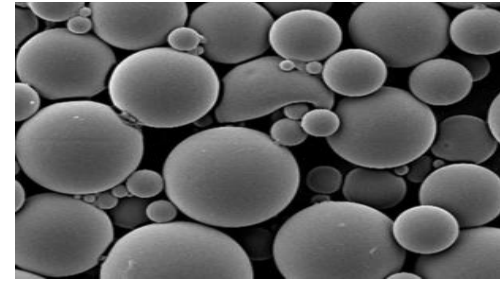


Microparticle

/ **INIBLANC** /



Fully solubilized PCL

Injection **30G** needle

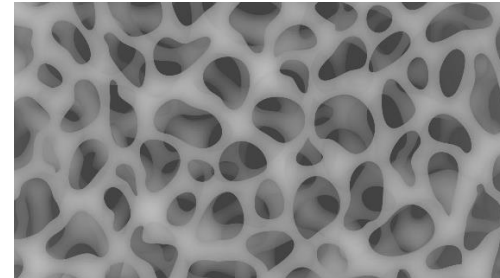

3D Matrix

| 3D Matrix formation | INIBLANC polymer forms 3D matrix in skin. It smoothly spreads into the entire skin.

INIBLANC PRODUCTS



Prefilled SYRINGE

Liquid Facelift
For under eyes

AMPOULE

Liquid Facelift
For full face


APPLICATION PART

/ UNIQUE TECHNOLOGY /

INIBLANC naturally spreads around the whole face.

No need for multiple injection points.

- INIBLANC Ampoule : Inject 0.2~0.25cc per point
- INIBLANC Syringe Poking point
- ✕ INIBLANC Syringe Target Injection Point : Inject 0.5cc per point

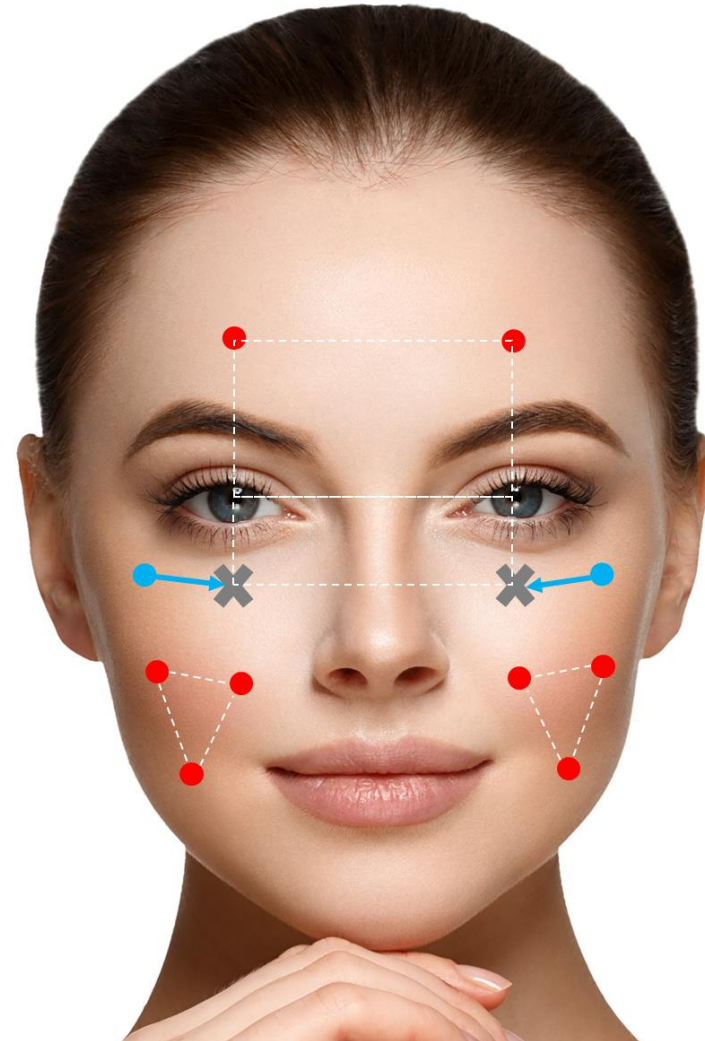

PROTOCOL




For the best results, 3 sessions are recommended at 2-4 week interval.
After the 3 sessions, apply additional 1~2 treatments at 3-6 month interval, depending on patient's condition.

IN-VIVO TEST

Collagen Rejuvenation in skin


CHANGE OF SKIN THINCKNESS

PBS
(Positive Control)

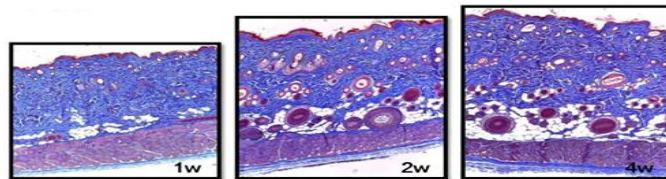


TIME / week →

INIBLANC

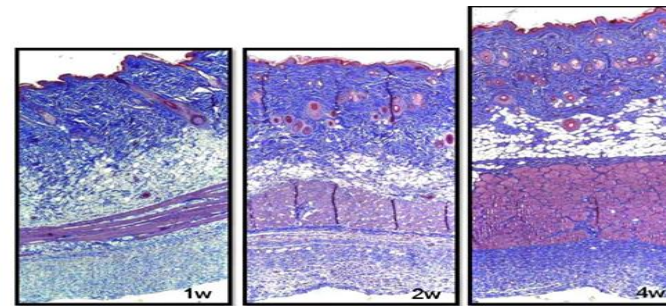


TIME / week →

CHANGE OF SKIN THINCKNESS IN DERMIS AND SUBCUTANEOUS LAYER

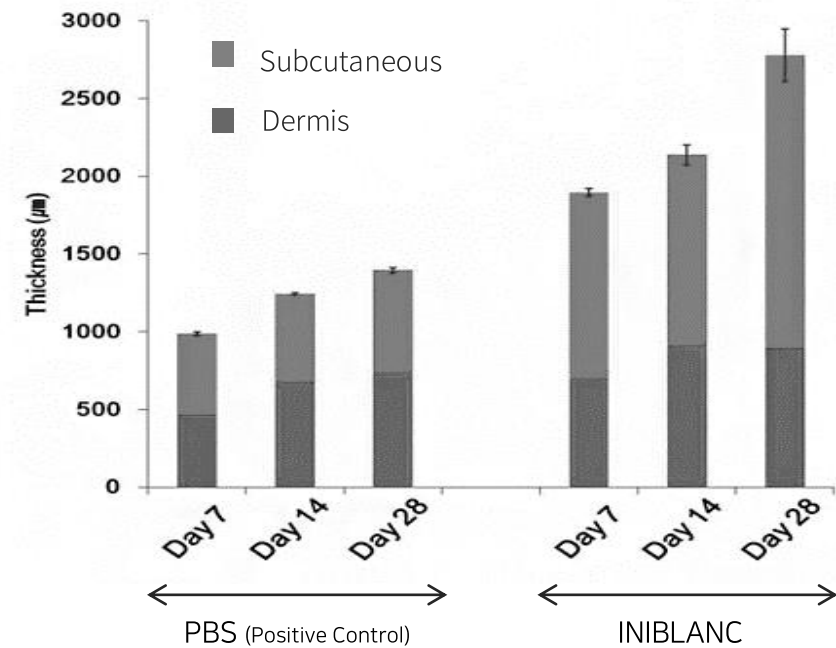

COLLAGEN SYNTHESIS RATE


Collagen Synthesis Rate by Age

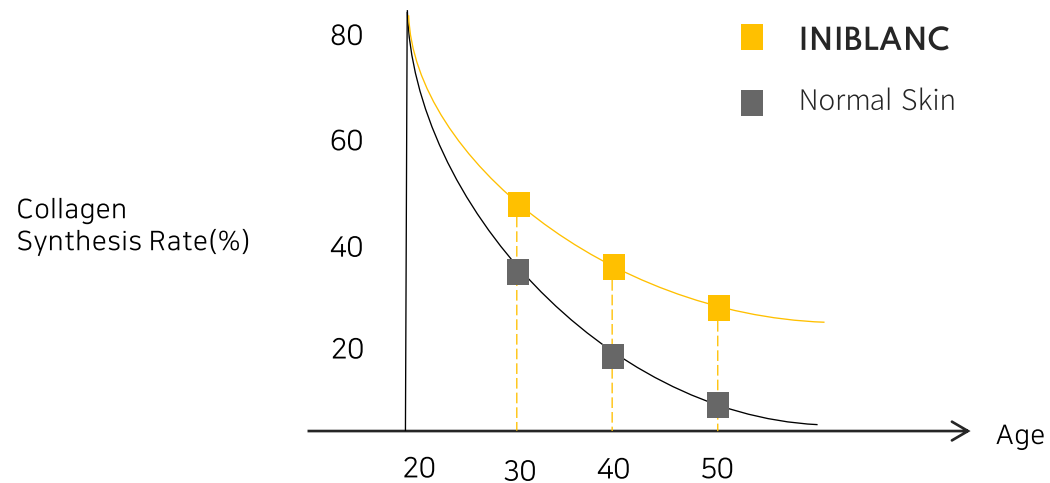

NIBL
Injectable Imp

-cha, 632;
u, Seoul, Korea
2-2136-6781

CHANGE OF SKIN DENSITY

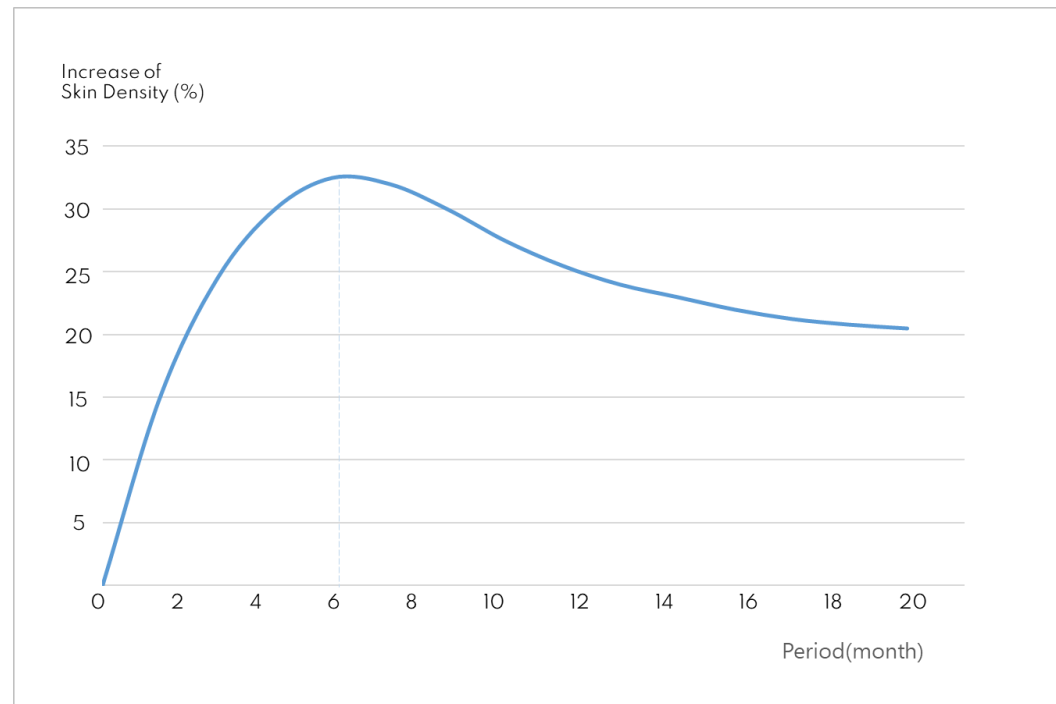

Change of Skin Density

The percentage of skin density increases and lasts up for a long time.


INIBL

Injectable Imp


5-cha, 632,
gu, Seoul, Korea
2-2-2136-6781

BEFORE & AFTER

/ AROUND EYES /

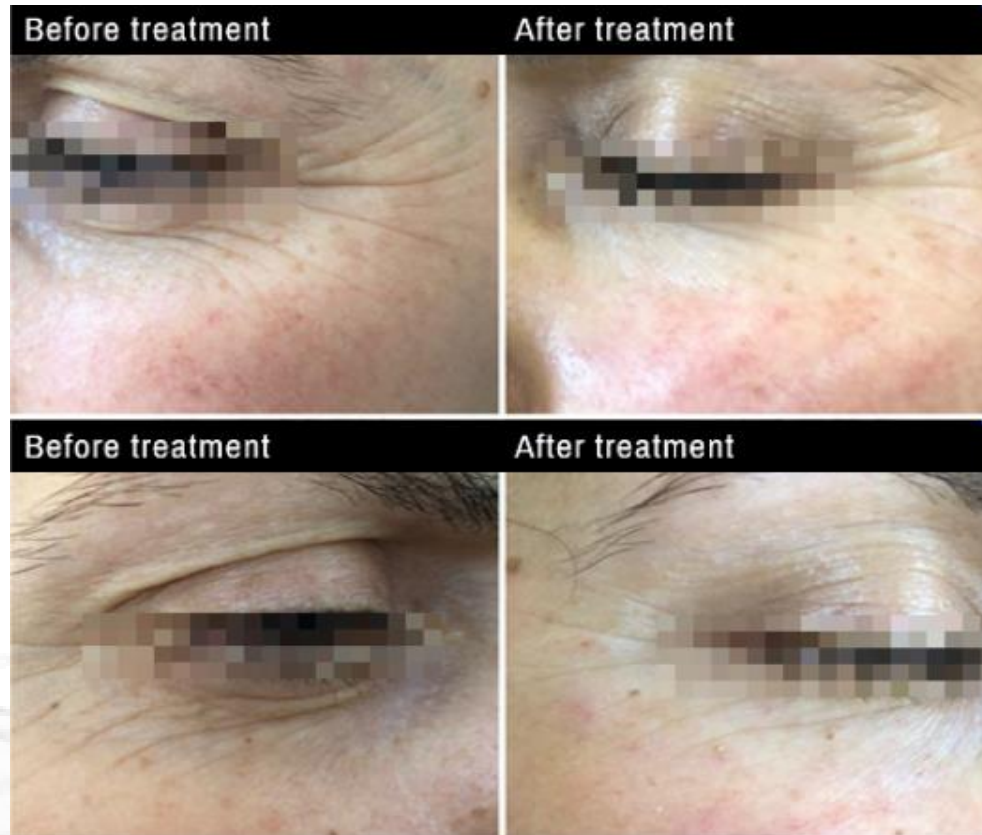

| 2 sessions at 4 weeks interval

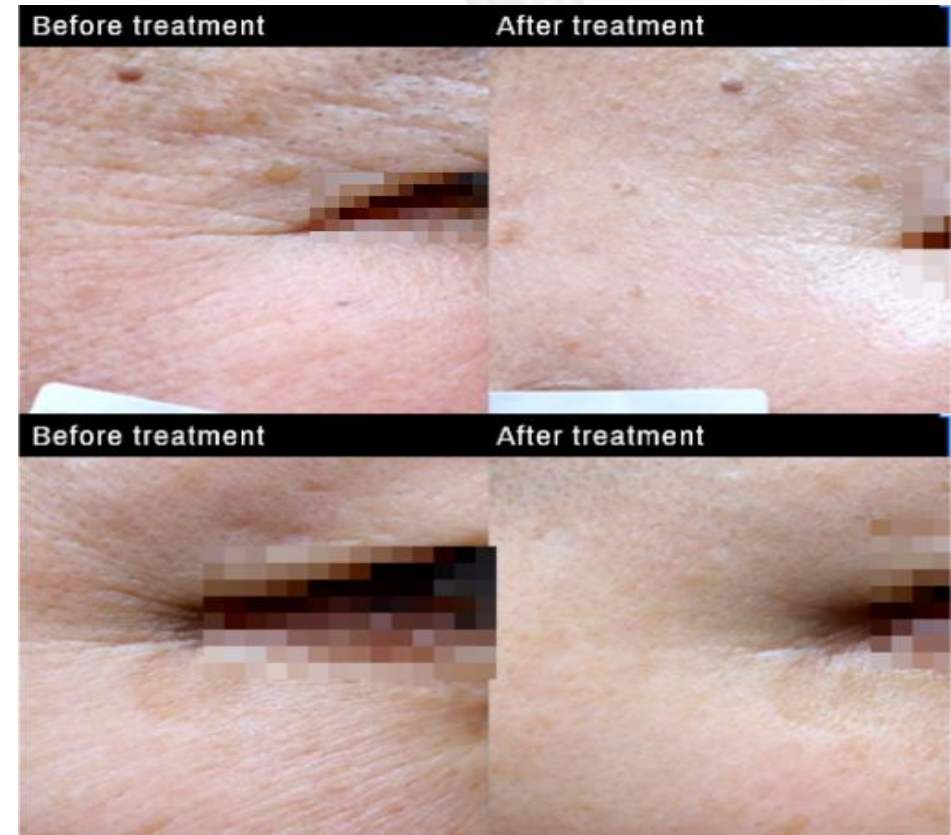


| 1 session after 12 weeks

BEFORE & AFTER

/FULL FACE/

BEFORE TREATMENT

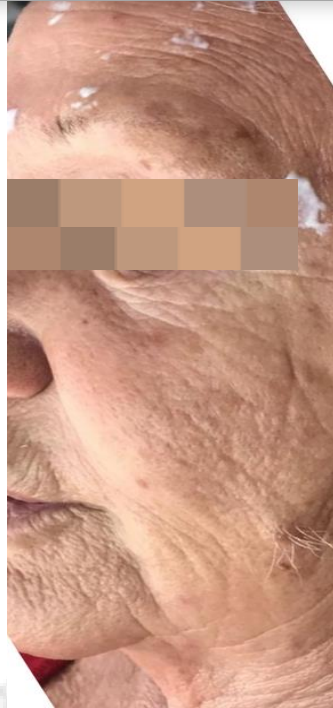

AFTER TREATMENT



| 4 sessions at 4 weeks interval